

NEW!

EZ CLICK™



Max Volume: 140mL

- Clicking lid for extra security
- All-in-one integrated cup
- 5 to 20 panels available
- Easy to use
- Read results at 5 minutes
- Built-in temperature strip (with optional adulteration strip)

DISTRIBUTED BY:

www.carolinachemistries.com


PROCEDURE

Step 1




Collect urine specimen and secure lid until it clicks.

Step 2

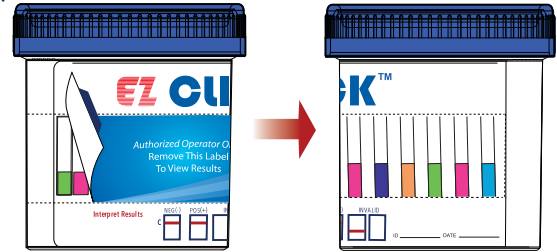

Peel off label to reveal test results.


Step 3

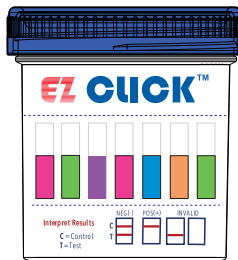

Read result at 5 minutes.

DO NOT INTERPRET RESULT AFTER 5 MINUTES.

INTERPRETATION OF RESULTS

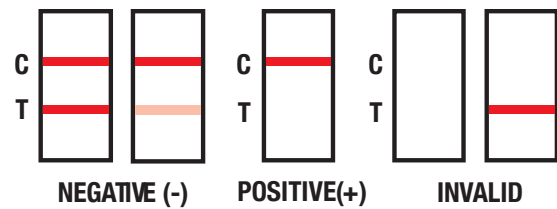

Preliminary Positive: One line appears in the control region (C). No line appears in the test region (T). The lack of a line in the test region (T) indicates a presumptive positive result for the corresponding drug of that specific test region.

Invalid: Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reason for control line failure.

The EZ CLICK™ Cup can test up to twenty (20) drug panels in one (1) device that can consist of any combination of drug analysis. This assay provides only preliminary qualitative test result. Use a more specific alternate quantitative analytical method to obtain a confirmed analytical result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method.

FOR *IN VITRO* DIAGNOSTIC USE ONLY

AMP AMP AMP	Amphetamine Urine Test*
BARB BARB BARB	Barbiturates Urine Test
BZO BZO BZO	Benzodiazepine Urine Test
COC COC COC	Cocaine (Benzoylecgonine) Urine Test*
THC THC THC	Marijuana (THC) Urine Test*
MDMA MDMA	MDMA (Ecstasy) Urine Test
MTD MTD MTD	Methadone Urine Test
MAMP MAMP	Methamphetamine Urine Test*
MOR MOR MOR	Morphine Urine Test
OXY OXY OXY	Oxycodone Urine Test
OPI OPI OPI	Opiate Urine Test
PCP PCP PCP	Phencyclidine Urine Test
TCA TCA TCA	Tricyclic Antidepressant Urine Test

BUP BUP BUP	Buprenorphine Urine Test
EDDP EDDP EDDP	EDDP Urine Test

*Lower cutoffs available for AMP500, COC150, MAMP500, THC20

PRODUCT DESCRIPTION & ORDERING INFO

PRODUCT NO.	DESCRIPTION	QUANTITY
EZ-154	DOA 5 AMP/COC/OPI/PCP/THC	25 TEST PER BOX
EZ-254	DOA 5 AMP/COC/mAMP/OPI/THC	25 TEST PER BOX
EZ-4254	DOA 5 AMP/BZO/COC/mAMP/OPI	25 TEST PER BOX
EZ-264	DOA 6 AMP/BZO/COC/mAMP/OPI/THC	25 TEST PER BOX
EZ-264 W/AD2	DOA 6 AMP/BZO/COC/mAMP/OPI/THC/SG-PH-CR	25 TEST PER BOX
EZ-564	DOA 6 BZO/COC/mAMP/OPI/OXY/THC	25 TEST PER BOX
EZ-4874	DOA 7 BZO/COC/mAMP/MDMA/OPI/OXY/THC	25 TEST PER BOX
EZ-3104	DOA 10 AMP/BARB/BZO/COC/mAMP/MDMA/MTD/OPI/PCP/THC	25 TEST PER BOX
EZ-4104	DOA 10 AMP/BARB/BZO/COC/mAMP/MTD/OPI/OXY/PCP/THC	25 TEST PER BOX
EZ-22104	DOA 10 BARB/BUP/BZO/COC/mAMP/MTD/OPI/OXY/TCA/THC	25 TEST PER BOX
EZ-47104	DOA 10 AMP/BUP/BZO/COC/mAMP/MDMA/MOR/MTD/OXY/THC	25 TEST PER BOX
EZ-2124	DOA 12 AMP/BARB/BZO/COC/mAMP/MDMA/MTD/OPI/OXY/PCP/TCA/THC	25 TEST PER BOX
EZ-9124	DOA 12 AMP/BARB/BUP/BZO/COC/mAMP/MDMA/MTD/OPI/OXY/PCP/THC	25 TEST PER BOX
EZ-9124 W/AD	DOA 12 AMP/BARB/BUP/BZO/COC/mAMP/MDMA/MTD/OPI/OXY/PCP/THC/PH-SG-OX	25 TEST PER BOX
EZ-9124 W/AD2	DOA 12 AMP/BARB/BUP/BZO/COC/mAMP/MDMA/MTD/OPI/OXY/PCP/THC/PH-SG-CR	25 TEST PER BOX
EZ-13124	DOA 12 AMP/BARB/BUP/BZO/COC/mAMP/MDMA/MOR/MTD/OXY/PCP/THC	25 TEST PER BOX
EZ-13124 W/AD	DOA 12 AMP/BARB/BUP/BZO/COC/mAMP/MDMA/MOR/MTD/OXY/PCP/THC/PH-SG-OX	25 TEST PER BOX
EZ-04144	DOA 14 AMP/BARB/BUP/BZO/COC/EDDP/mAMP/MDMA/MTD/OPI/OXY/PCP/TCA/THC	25 TEST PER BOX
EZ-04144 W/AD	DOA 14 AMP/BARB/BUP/BZO/COC/EDDP/mAMP/MDMA/MTD/OPI/OXY/PCP/TCA/THC/PH-SG-OX	25 TEST PER BOX
EZ-04144 W/AD2	DOA 14 AMP/BARB/BUP/BZO/COC/EDDP/mAMP/MDMA/MTD/OPI/OXY/PCP/TCA/THC/PH-SG-CR	25 TEST PER BOX
EZ-06144	DOA 14 AMP/BARB/BUP/BZO/COC/EDDP/mAMP/MDMA/MOR/MTD/OXY/PCP/TCA/THC	25 TEST PER BOX
EZ-06144 W/AD	DOA 14 AMP/BARB/BUP/BZO/COC/EDDP/mAMP/MDMA/MOR/MTD/OXY/PCP/TCA/THC/PH-SG-OX	25 TEST PER BOX
EZ-06144 W/AD2	DOA 14 AMP/BARB/BUP/BZO/COC/EDDP/mAMP/MDMA/MOR/MTD/OXY/PCP/TCA/THC/PH-SG-CR	25 TEST PER BOX

FREQUENTLY ASKED QUESTIONS

What does DOA stand for?

It stands for Drug of Abuse.

What does AD stand for?

AD stands for Adulterants. Adulterants are used in order to check if the test samples have been contaminated/altered.

What is the shelf life on the EZ CLICK™ Cup?

24 months from date of manufacture when kept at room temperature (39° F-86° F).

Do EZ CLICK™ cups come with a temperature strip?

Yes. All EZ CLICK™ cups come with a temperature strip.

Are there built-in controls?

Yes. There is a built-in control region on the test strips. For facilities that require QC testing, EZ CLICK™ has control sets (+/-) available for purchase (Part No.: D0115-TC01 and D0115-TC02)

Are EZ CLICK™ cups CLIA Waived?

Yes. See 'Ordering Info' on page 3 to find which cups are CLIA Waived.

Do we custom make configurations?

Yes we do. It may take up to eight weeks depending on the configuration. Certain conditions may be applied.

How should the product be disposed?

The device itself may be disposed as solid waste. Device tested with a donor's sample should be handled as potentially bio-hazardous material in accordance with federal, state, and local regulations.

When should urine be collected for the test?

Urine from any time of day can be used. The minimum detection time varies for different drugs. (Refer to the approximate detection timetable below).

What is the EZ CLICK™ Cup?

The EZ CLICK™ Cup is a rapid test for qualitative detection of Amphetamine, Cocaine, Marijuana, Oxazepam, Methamphetamine, Opiate, Methadone, Oxycodone, Phencyclidine, Butalbital, Buprenorphine, Morphine, EDDP, MDMA, and Nortriptyline, in human urine. The EZ CLICK™ Cup yields a positive result when a drug and/or its metabolite in urine is at or exceeds its cutoff concentration.

What is a false positive test?

The definition of a false positive test would be an instance where the test result from the EZ CLICK™ Cup is positive, even though the initial target drug is not present in the sample. The most common causes of a false positive test are cross reactants. certain foods and medicines, diet plan drugs, and nutritional supplements may also cause a false positive test result with this product.

What is a false negative test?

The definition of a false negative test is that the initial target drug is present but is not detected by EZ CLICK™ Cup. If the sample is diluted, or if the sample is tainted or contaminated with a substance this could cause false negative results.

What is the test's limitation?

The EZ CLICK™ Cup provides only a qualitative, preliminary analytical result. A secondary analytical method must be used to obtain a confirmed result. Gas chromatography/mass spectrometry (GC/MS) is the preferred confirmatory method

CUT-OFF VALUE AND APPROXIMATE DETECTION TIME

DRUG	DRUG ABBREVIATION	CUTOFF LEVEL	MINIMUM DETECTION TIME	MAXIMUM DETECTION TIME
Amphetamine	AMP	1000ng/mL	4-6 hours	2-3 days
Butalbital	BARB	300ng/mL	2-4 hours	1-3 weeks
Buprenorphine	BUP	10ng/mL	2-6 hours	2-4 days
Oxazepam	BZO	300ng/mL	2-7 hours	1-4 days
Cocaine	COC	300ng/mL	2-6 hours	2-3 days
EDDP	EDDP	300ng/mL	3-8 hours	1-3 days
Methamphetamine	mAMP	1000ng/mL	4-6 hours	2-3 days
MDMA	MDMA	500ng/mL	2-7 hours	2-4 days
Morphine	MOR	300ng/mL	2-6 hours	1-3 days
Methadone	MTD	300ng/mL	3-8 hours	1-3 days
Opiate	OPI	2000ng/mL	2-6 hours	1-3 days
Oxycodone	OXY	100ng/mL	1-3 hours	1-2 days
Phencyclidine	PCP	25ng/mL	4-6 hours	7-14 days
Nortriptyline	TCA	1000ng/mL	8-12 hours	2-7 days
Marijuana	THC	50ng/mL	1-3 hours	1-7 days

HEMOSURE, INC.®
5358 Irwindale Ave., Irwindale, CA 91706, USA
T: 626.443.8480 • F: 626.443.8065 • sales@hemosure.com
www.hemosure.com

DISTRIBUTED BY:



contactsales@carolinachemistries.com
carolinachemistries.com

877-722-8910

