

Status

Flu A & B



An *in vitro* rapid qualitative test that detects influenza type A and type B antigens directly from nasal swab, nasopharyngeal swab and nasal aspirate/wash specimens*

FLU FEARS US. YOU WILL LOVE US!



THEY CAN RUN BUT CANNOT HIDE

- Meets the new FDA reclassification performance criteria
- CLIA WAIVED for swab specimens
- Includes Positive and Negative control swabs
- Innovative flip design with onboard sample extraction
- Differentiates between Influenza A and B
- Pre-measured developer solution capsule for increased accuracy and ease of use
- Flocked nasal swabs for improved patient comfort and superior specimen collection

ORDERING INFORMATION

PRODUCT	# OF TESTS	ITEM #
Status Flu A & B	25	36025
Nasal Aspirate Kit	25	510AS

DISTRIBUTED BY:



contactsales@carolinachemistries.com
 carolinachemistries.com
 877-722-8910

Proudly made in the USA

For more information or to place an order, contact:




A PBM Group Company


85 Orchard Road • Skillman • New Jersey • 08558
 800.526.2125 • 732.246.3366 • Fax: 732.246.0570 • www.lifesignmed.com

*nasal aspirate/wash specimens – non waived

Simple Procedure for Swab Specimens

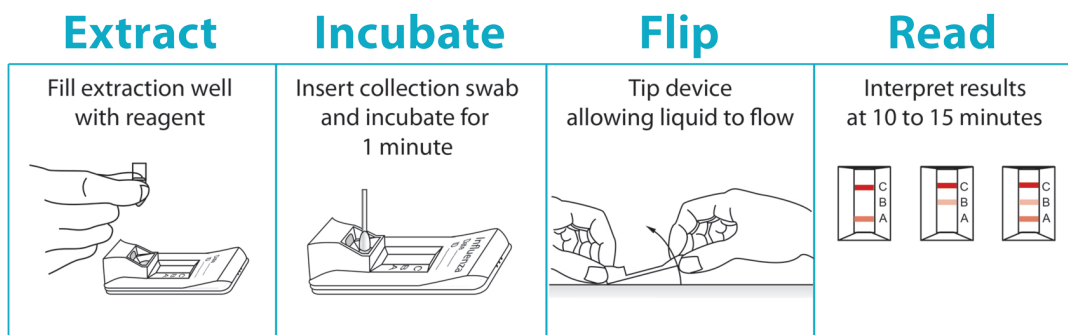

**See Package Insert For Details

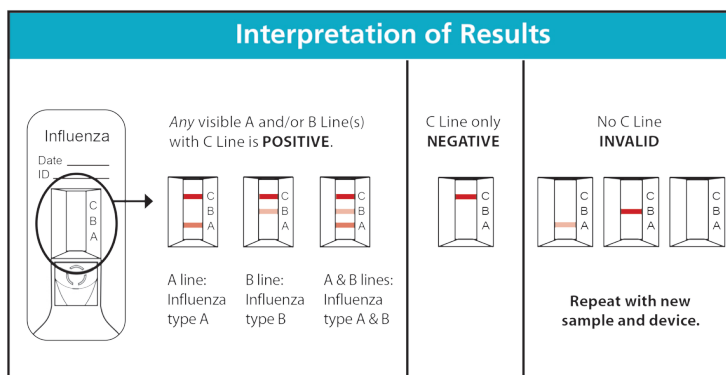

Proven Clinical Performance

Status Flu A & B vs. PCR

	Influenza A	Influenza B
Positive Percent Agreement (PPA)	89.2 % (95% CI: 83.0 - 93.4%)	86.4 % (95% CI: 80.1 - 90.9%)
Negative Percent Agreement (NPA)	99.4 % (95% CI: 97.7 - 99.8%)	99.0 % (95% CI: 97.1 - 99.7%)

Wide Influenza strain coverage: 49 strains including A/CA/2009(H1N1v), A/Minesota/2010(H3N2v), H5NX and H7N9.

CPT CODE

INFLUENZA A 87804QW

INFLUENZA B 87804QW-59