

Status Strep A Plus

Rapid immunochromatographic assay for the detection of group A streptococcal antigen directly from throat swab specimens



- **CLIA WAIVED**
- Simple procedure
- Easy extraction for fast reaction times
- Results in 5-10 minutes
- Sensitivity of 96.2% and specificity of 98.7%
- External negative and positive controls included
- New eco friendly and easy to use packaging 🌿

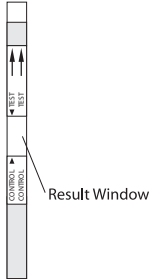
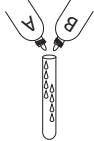
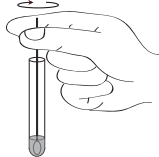


CPT CODE

Status Strep A Plus

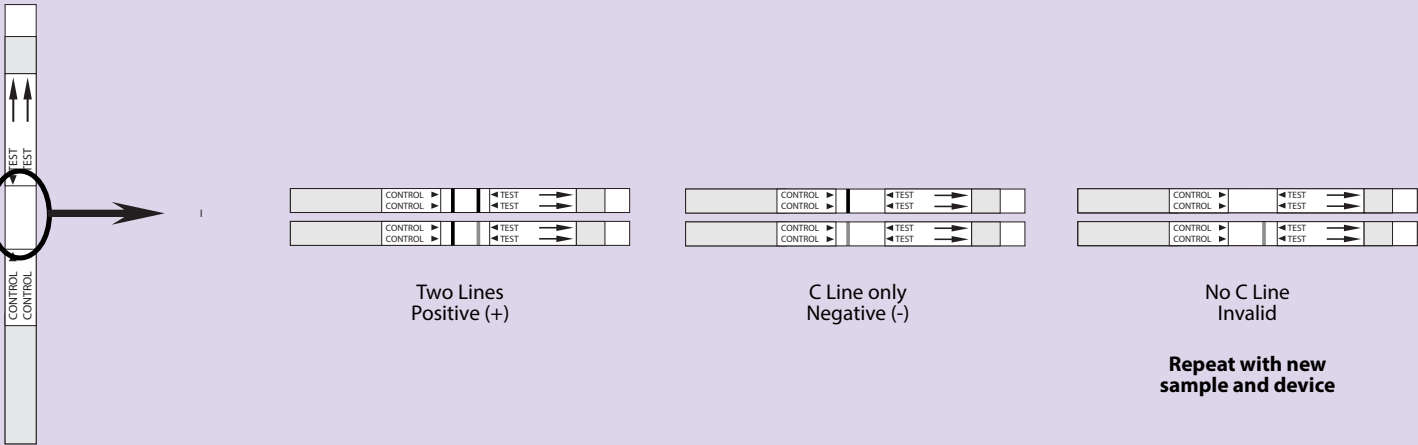
87880QW

Status Strep A Plus

Refer to Product Instructions for details.

TEST DEVICE	PROCEDURE			
 <p>Result Window</p>	<p>1. Add 4 drops each of reagent A and reagent B into extraction tube and mix by gently swirling.</p> 	<p>2. Place specimen swab into extraction tube, rotate vigorously with extraction solution and let stand for 1-2 minutes.</p> 	<p>3. Squeeze out as much liquid as possible by pressing firmly against the side of the tube with two fingers, discard the swab.</p> 	<p>4. Remove the Status Strep A test strip from the sealed pouch and insert into the tube of extracted solution and allow the migration to begin. Read Results in 5 minutes but not after 10 minutes</p> 

Interpretation of Results



Two Lines Positive (+)

C Line only Negative (-)

No C Line Invalid

Repeat with new sample and device

Also Available



- Status Strep A Flip
- Status Strep A Strip
- Status Accustrep

ORDERING INFORMATION

PRODUCT	CLIA COMPLEXITY	# OF TESTS	ITEM #
Status Strep A Plus	Waived	50	34250



 Proudly made in the USA

For more information or to place an order, contact:

DISTRIBUTED BY:



contactsales@carolinachemistries.com
carolinachemistries.com

877-722-8910



A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732.246.3366 • Fax: 732.246.0570 • www.lifesignmed.com

Printed in USA.
All rights reserved.
582-3/23/12