

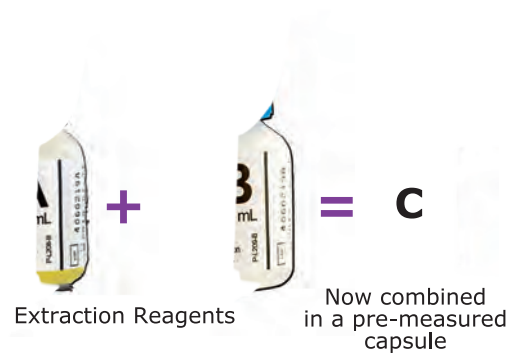
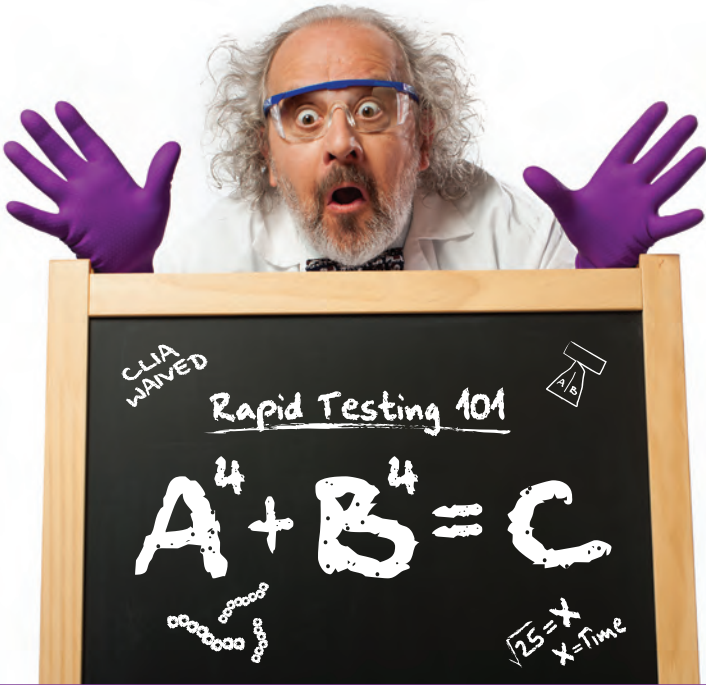
Status

Strep A-Flip cassette

Our New One **STREP** Solution!

Status Strep A-Flip

- **CLIA WAIVED**
- Patented single use reagent capsules
- Positive results in 5 minutes
- Sensitivity of 96% and specificity of 99%
- No counting drops – less steps for greater accuracy



Kit Contents

- 25 swabs
- 25 test devices
- 25 reagent capsules
- Positive & negative external controls

ORDERING INFORMATION

PRODUCT	CLIA COMPLEXITY	# OF TESTS	ITEM #
Status Strep A-Flip (Cassette)	Waived	25	34125

For more information or to place an order, contact:

DISTRIBUTED BY:



contactsales@carolinachemistries.com
carolinachemistries.com

877-722-8910



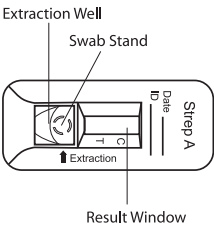

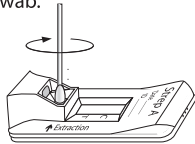
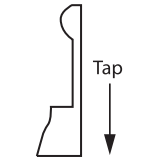
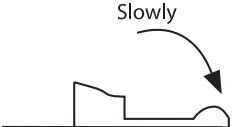
A PBM Group Company

85 Orchard Road • Skillman • New Jersey • 08558
800.526.2125 • 732.246.3366 • Fax: 732.246.0570 • www.lifesignmed.com

Status Strep A-Flip cassette

Rapid Detection of Group A Streptococcal Antigen
Directly From Throat Swab Specimens.

Status Strep A-Flip Refer to package insert before performing a test

TEST DEVICE	PROCEDURE			
	<p>1. Twist the tab off the Extraction Capsule. Squeeze the entire contents of the Extraction Reagent Capsule into the extraction well of the test device.</p> 	<p>2. Place specimen swab into extraction well and spin 5 times to mix specimen. Let incubate for 1-2 minutes and spin 5 more times. Remove swab.</p> 	<p>3. Raise the device upright, tap device on a flat surface to ensure the specimen flows into the hole.</p> 	<p>4. Slowly lay the device back down onto flat surface and read results after 5-10 minutes.</p> 

CPT Code 87880QW