

PHONE: 781-275-4892

FAX: 781-275-2731

DATE: 16-March-2026

BULLETIN #: B2603

PRODUCTS: EasyRA EasyQC® Quality Control Material Level A Lot# 25008

EasyRA EasyQC® Quality Control Material Level B Lot# 25009

SUBJECT: Revised AST QC Ranges

An internal review of field data and Medica testing for EasyQC® Quality Control Level A, REF 10793, Lot# 25008 and Level B, REF 10794, Lot# 25009, used with the EasyRA® clinical chemistry analyzer, determined that the AST ranges require revision. Patient results are **not** affected.

Revised AST QC Ranges

REF: 10793

Product: EasyQC Level A

Lot#: 25008

Expiration: 10/2027

Test	Published Range (U/L)	Revised Range (U/L)
AST	46.4 (40.4 – 52.4)	43.8 (37.8 – 49.8)

REF: 10794

Product: EasyQC Level B

Lot#: 25009

Expiration: 11/2027

Test	Published Range (U/L)	Revised Range (U/L)
AST	174.6 (159.6 – 189.6)	164.4 (149.4 – 179.4)

To update the QC ranges on the EasyRA® clinical chemistry analyzer refer, to Chapter 4 “Setting Up QC Materials Manually” in the EasyRA® Operator’s Manual.

We apologize for any inconvenience and appreciate your continued support.

For questions, please contact Medica Technical Support:

Phone: (781) 275-4892

Email: EasyRAtech@medicacorp.com